



Techno-Aide, Inc.
7117 Centennial Boulevard
Nashville, TN 37209
P: 800.251.2629
F: 800.554.6275

Mobile Receptor Holders CLEANING & CARE INSTRUCTIONS 2018

Thank you for choosing Techno-Aide for your radiology accessory needs. To ensure the longevity of your new mobile receptor holder, we recommend the following guidelines for cleaning and care.

Cleaning:

We all know that in a medical environment, it is easy for positioning aides to become dirty and soiled from blood, bodily fluids, betadine, barium, and other fluids.

Do:

- Techno-Aide recommends cleaning holder with cold water and a mild detergent to remove all immediate contacts with fluids. 409 Spray can also be used to clean the surface to remove dust & dirt from the holder.
- To disinfect your holder, pole, and base, Techno-Aide recommends the use of disinfectant wipes such as PDI Sani-Cloth Germicidal Wipes Cat# GWP-WAF. Allow a drying time of at least 3 minutes.

Care and Replacement Parts:

Techno-Aide provides a one-year warranty on workmanship and materials for our radiology accessory products. Care should be taken when moving the holder up and down the pole for positioning. Holder will move up and down the pole with a spring-pin. Should you need a replacement part, please contact Techno-Aide customer service 1-800-251-2629.