


Components and Assembly

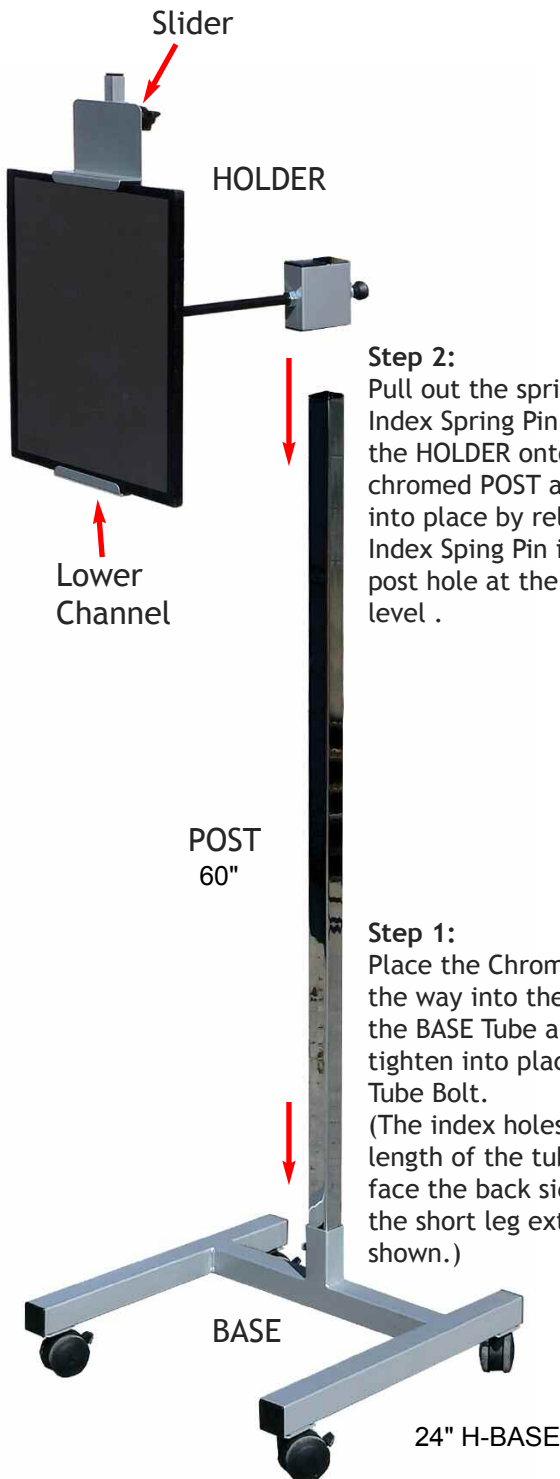

Step 2: Pull out the spring-loaded Index Spring Pin and slide the HOLDER onto the chromed POST and lock into place by releasing the Index Spring Pin into the post hole at the desired level.

Step 1: Place the Chromed POST all the way into the bottom of the BASE Tube and securely tighten into place with the Tube Bolt. (The index holes down the length of the tube should face the back side toward the short leg extensions as shown.)

Instructions for Use


MCH-33B

(standard tilt & rotate)

(a) Move HOLDER up and down on chrome post to desired position with the Index Spring Pin-a.

(b) Place the receptor onto Lower Channel and position the Slider onto the top of the receptor and secure with Knob-b.

(c) Loosen Knob-c and rotate the HOLDER into the desired position and re-tighten Knob-c.

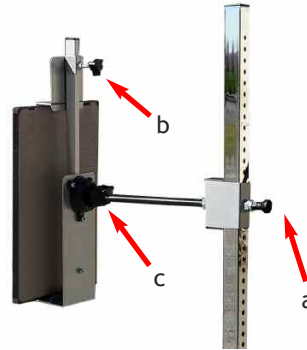


MCH-37B

(extension arm tilt & rotate)

(a), (b), (c) See MCH-33 above.

(d) Pull the Holder away from the Post to extend the Holder closer the patient. Push the Holder back to the Post when not extended and it should gently lock into place.

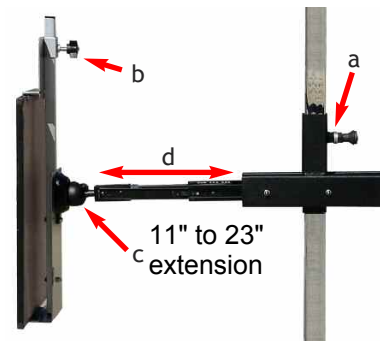

Channel is 1 1/4"


MCH-39B

(tall receptor holder)

(a) Move HOLDER up and down on chrome post to desired position with the Index Spring Pin-a.

(b) Place receptor onto Lower Channel and position the Slider a little over halfway up of the receptor and secure with Knob-b.

(c) Move the lateral sliders onto the sides of the receptor and tighten Knob-c (x2).

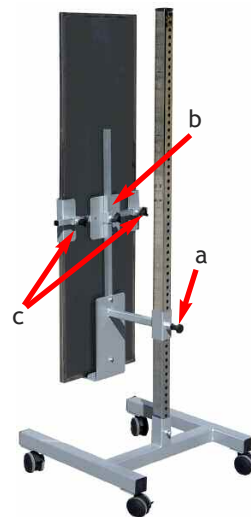

MCH-40B

(DR-Receptor holder)

(a) Move HOLDER up and down on chrome post to desired position with the Index Spring Pin-a.

(b) Place DR-Receptor onto Lower Channel and position the Slider onto the top of the plate and secure with Knob-b.

